If you stand the bath bench upright and realize it is facing the wrong way when you put it in the tub, lay it back on its side (as in step 4) and remove the back. Then reinstall the back from the opposite side.

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## Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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## Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number

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3

**ATTACH THE ARM SUPPORT - FIXEZ L'APPUI-BRAS**

Slide and click the arm support into the seat receivers.  
Note: Arm support should slant outwards slightly

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