

# EZee Life™

## Folding Wheeled Commode

Model: CH1025



EZee Life™ Products  
34 Futurity Gate, Unit 15  
Concord, Ontario L4K 1S6



## Specifications

Weight capacity:	220 lbs	(100 kgs)
Colour:	White PE	
Seat Height:	18 1/2"	(47 cm)
Seat Width:	17"	(43 cm)
Seat Depth:	15 3/4"	(40 cm)
Backrest Height:	12 1/4"	(31 cm)
Overall Width:	21 1/2"	(55 cm)
Weight:	11.5 lbs	(5.2 kg)

## Important Information:

1. Read the instructions before assembling; do not use this product without proper instruction from a professional.
2. If components are damaged or missing, contact your dealer immediately.
3. Do not transfer to or from this commode unless the casters are locked
4. Always check for secure fit of raised toilet seat.
5. Check for damage or excessive wear before each use.

## Intended Use:

To provide support over a toilet, as a bedside toilet.

## Daily Maintenance:

- Always empty the sanitary bucket after using
- Clean Toilet seat and bucket
- Clean the upholstery and frame with a moist cloth
- Use the toilet detergents according to instructions given by a manufacturer
- Remove contaminants like threads or hair off the movable parts of the wheelchair (wheels, footrests).

Keeping all parts of the commode clean and using it properly will lengthen its lifespan and provide years of trouble free service. Storing the commode in a moist place (steam baths, shower rooms) or outside together with a lack of hygiene can lead to excessive wear, malfunctions and material degradation.

## Commode Assembly:

- Attach the casters by inserting the caster stem into the commode frame; attaching the wheels equipped with brakes at the back of the commode and the non-braked wheels in the front.
- Install the sanitary bucket under the seat on bucket frame

## Commode Parts:

- 1) Armrest
- 2) Side Frame
- 3) Seat Support Bar
- 4) Sanitary Bucket
- 5) Caster
- 6) Caster Brake
- 7) Bucket Frame
- 8) Commode Seat
- 9) Seat Lid



## Instructions:

- 1) The commode can be used to transport the user very short distances but it is not meant to be a transport device. Use for short trips only.
- 2) To use in place of a toilet, slip the sanitary bucket into its bracket under the seat.
- 3) To use over a toilet, remove the sanitary bucket, if and back the commode over the toilet.
- 4) To fold, remove sanitary bucket, move side frame outward and allow seat support bar to fold down. Fold two side frames into the center.





## Safety Information:

- This commode is designed for indoor use, assistance of a caretaker is necessary to use.
- The commode cannot be used to transport users over architectural barriers (steps, driveways, ramps).
- Do not use the commode on soft, uneven, unprotected surfaces such as grass, gravel or sand.
- Using the commode on slopes with a grade higher than 2% is dangerous – risk of falling off the wheelchair.
- The commode cannot be used as seat in motor vehicles.
- In order to improve user's safety, you can attach a safety belt.
- Do not stand on the seat of the commode
- Do not lean excessively in any direction when sitting on the commode

**Caution:** It is dangerous to use the commode after discovering any damage or when the wear of some parts may cause an accident. All the repairs need to be performed by skilled personnel.

## Inspections:

The commode should be inspected every three months or more often as needed. During the inspection one needs to:

- Check the brakes for damage or wear and ensure they work securely
- Check for loose or missing screws and hardware
- Check wheels for wear or damage and ensure they rotate freely (a grinding noise may indicate bearings are worn and need adjustment or replacement)
- Check frame joints for cracks or damage

## **Transfers:**

In order to transfer to and from the commode the brakes must be locked and a caregiver should be available to help.

The user can do a standing transfer from a bed, chair or wheelchair using the armrests for support and balance and removing or swinging way the footrests.

Alternatively the user can remove one arm and transfer onto the commode seat using a sliding transfer style.

For users who cannot assist in transferring or need a lot of help various types of lifts can be used for transfers.

## **Warranty:**

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the item. Please contact your supplier/dealer for assistance should a defect become apparent.

The warranty does not extend to consequential costs resulting from the fault or defect in the product, in particular freight and travel costs, loss of earnings, expenses or other expenses which may occur.

The warranty does not cover normal wear and tear resulting from normal use or damage caused by accident, misuse, negligence or improper installation or assembly.

The vendor/manufacturer will not accept responsibility for any damages or injury caused by misuse, negligence or non-observance of the instructions supplied.

## **Imported & Distributed by:**

Factory Direct Medical  
34 Futurity Gate, Unit # 15  
Concord Ontario L4K 1S6  
Local: 416-739-8393  
Toll Free: 855-235-2400

