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
97iJ 99Z3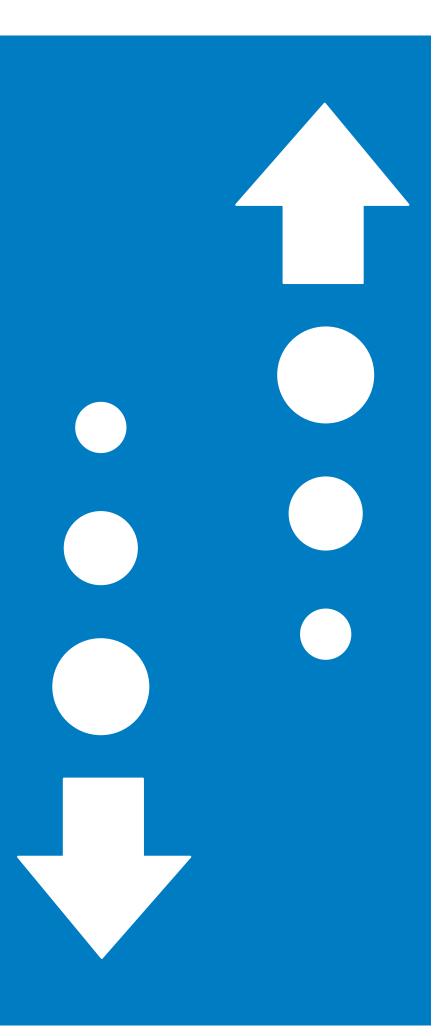

EZee Life

Aluminum Crutches Push-Button/Adjustable Fully Assembled

Item # MFG # CRP0105 CH2030

Limited Warranty

EZee Life brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your **EZee Life** device or this warranty, please contact an authorized **EZee Life** dealer.

EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

EZee Life

Adjusting Handle Position

- 1. Do one of the following:
 - remove the wing nut and bolt securing the handle to the crutch (Detail "A") or
 - push both handle locks IN (direction of arrow stamped on side of handle lock) (Detail "B")
- 2. Align the handle with the desired set of adjustment holes.


Note: Quick Adjust Crutch – There will be an audible click when handle locks are fully engaged and secure.

- 3. If necessary, secure with wing nut and bolt.
- 4. Repeat for other crutch.


Adjusting the Leg Extension

Depress both snap buttons on the leg extension and allow them to snap through the desired adjustment holes on the crutch leg frame. Repeat for other crutch (Detail "C").


Detail "A" - Quick Change Crutch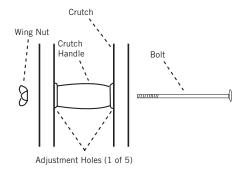

Detail "B" - Quick Adjust Crutch

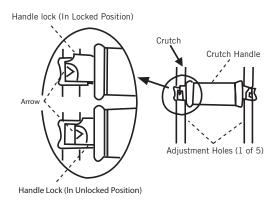
Detail "C" Leg Extension

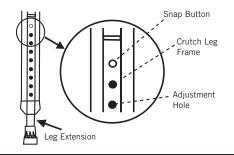CAUTIONS:

- Use after proper training under the direction of your doctor or physical therapist
- Replace crutch tips if worn
- Keep hand grip wing nut tightened
- Use extra caution on wet and slippery surfaces
- Highest setting, armrest to floor: 45" 53"
- Patient height range: 5'2"- 5'10"
- Supports up to 300 Lbs

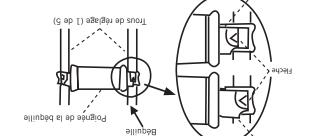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