# EZee Life

## Aluminum Round Bath Stool

Item #  
CRP0216

Mfg #  
CH2040

### Limited Warranty

EZee Life brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

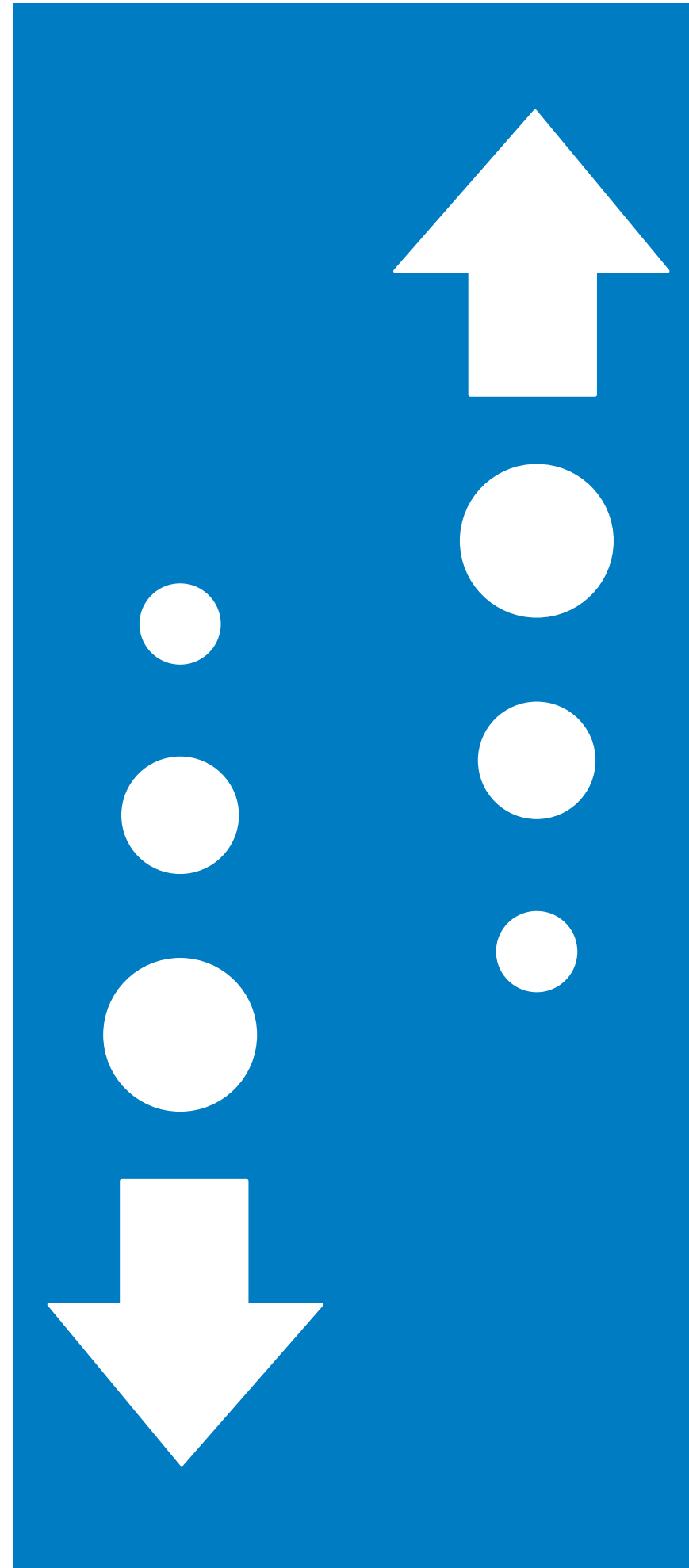
In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your EZee Life device or this warranty, please contact an authorized EZee Life dealer.

EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

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EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

# EZee Life

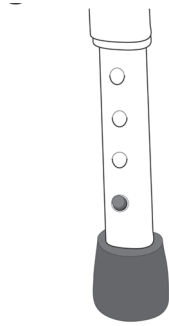
## Features:

- Manufactured with 1" aluminum tubing to ensure lightweight stability
- Impact resistance, composite seat is crack-proof and tarnish resistant
- Seat height from: 13" – 20"
- Seat width: 13"
- Width of legs: 13.5
- Weight Capacity: 300 LBS

## Assembly Instructions

1. Bath Stool comes fully assembled. You can adjust the height of bath stool by depressing pins.
2. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per (figure 1).
3. To be sure legs are locked into place by pushing down on or by pulling legs.

## Care and maintenance

- To maintain the beauty of the original finish, clean with mild soap and rinse with warm water.
- Wipe bath stool dry after each use.

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