

# EZee Life™

## “X” Frame Folding Rollator

Models: CH3011 (Black)  
CH3012 (Red)  
CH3013 (Blue)



EZee Life™ Products



## Specifications

- Frame - Lightweight Aluminum
- Weight – 7.6 kg (26.7 lbs)
- Casters – 20 cm (7.9")
- Seat Width – 47 cm (18.5")
- Seat Depth – 25 cm (10")
- Seat Height – 53 cm (20.9")
- Back Height – 20.3 cm (8")
- Weight Capacity – 130 kg (285 lbs)

## Features:

- Black, Red or Blue Folding Frame
- Lightweight Aluminum
- 8" Backrest
- Mesh Sling Seat for easy folding
- Adjustable Handle Height
- Locking Brakes
- Fabric Carry Bag
- Cup and Cane Holder

## Limited Warranty:

EZee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. EZee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

## Description

The EZee Life™ rollators are manufactured from high quality materials and include seats, fabric carry bag, locking brakes and adjustable height push handles. This rollator folds in two directions for easy transport and storage.

## Intended Use

Designed to assist the user to walk, rest and carry small items when shopping for both indoor and outdoor use on the flat surfaces by a single individual with limited mobility with a maximum weight of 130 kg (285 lbs).

The intended user should possess the necessary physical, visual and cognitive skills to operate the rollator safely. If you have any doubt, please consult your dealer/prescriber.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

## Safety Advice

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handlebars with the unit in front of the user. Read this manual carefully before using your rollator

- 1) Always ensure the brakes are in the locked position when unattended.
- 2) Always lock the brakes before sitting on the seat.
- 3) Check all casters and wheels are secure before use.
- 4) Check the brakes are functioning properly.
- 5) Check handle height adjustment hand knobs are secure before use.
- 6) Check the rollator is fully unfolded.
- 7) Take care when carrying the rollator when folded.
- 8) Keep your hands away from the folding mechanism to avoid pinching risk.
- 9) DO NOT drop down onto the seat. Sit in a controlled manner.
- 10) DO NOT hang bags or other objects from brakes or push handles to avoid stability and brake problems
- 11) DO NOT use the rollator as a transportation device. The seat is intended for resting only and the brakes must be engaged when the seat is in use.





## Components

- 1 Backrest
- 2 Adjustable Parking/Running Brake
- 3 Seat Area
- 4 Height Adjustable Handle
- 5 Caster 200mm (7.9")
- 6 Shopping Bag
- 7 Cane Holder
- 8 Rear Leg Folding Knob



## Assembly Instructions

1. Undo hook & loop, remove rear wheel folding mechanism knob.
2. Rotate the rear wheel arms backward.
3. Insert and tighten rear wheel folding mechanism knobs firmly, ensuring proper alignment.
4. Remove front wheel folding mechanism knobs.
5. Rotate the front wheel arms forward.
6. Insert and tighten front wheel folding mechanism knobs firmly, ensuring proper alignment.
7. Pull the two push handles apart.

8. Open the frame until the crossbar locking mechanism locked automatically (as shown below), verify seat is locked in place by pushing down on both seat edges.




## Handle Height Adjustment

The rollator push handles are adjustable height for comfort and ease of use.

- ! Depress the push pin,
- ! Adjust the handle to the desired height
- ! Allow the push pin to protrude back through the closest adjustment hole.

**Warning:** Ensure the push pin fully pops through the frame adjustment hole.

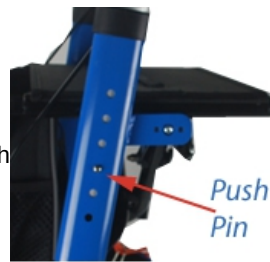

## Walker Handle Adjustment Guide

When the handle height of the walker is adjusted properly, the user can stand with a normal posture while wearing the footwear they most commonly wear.

Adjusting the rollator walker height is easily done without the need of tools. There will be a knob where the handles meet the frame of the walker which when loosened and removed will allow the handles to be raised and lowered. The seat height of the rollator walker will usually be fixed at a certain height and not adjustable.

While the user is standing with their arms hanging limply at their sides, the distance from the inside of their wrist (where a snug watchband would normally be) to the floor should be equal to the top of the walker handles to the floor.


## Brake Operation & Adjustment

1. Pull up on brake handle lever to slow the rollator
2. Push down on brake handle lever to lock the rear wheels. Pull up on the brake lever handle to release the lock



- 3. Brakes must be in the lock position when seat is going to be used. Ensure the brakes are locked by pushing the rollator.
- 4. Brake adjustment is carried out as following:

Turn the adjuster screw anti-clockwise to increase the tension then lock the adjustment by turning the Locking Nut clockwise while holding the adjustment screw stationary stationary.

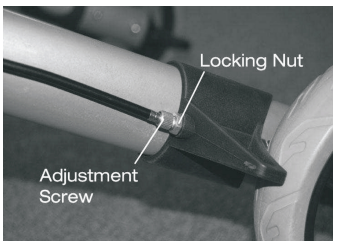

***Bake adjustment reduced to allow use with too light a pressure may be ineffective in locking the brakes and preventing the rollator's movement.***

### Folding

- Remove the accessories and lower down the handles if necessary.
- Lift up the crossbar locking mechanism release belt on the mesh seat to release the crossbar locks.
- Fold the frame as shown on right.
- Push the two push handles together.
- Release the front and rear wheel folding mechanism knobs and rotate the rear wheel arms inward, replace folding mechanism knobs and attach hook and loop to rear legs.

### Attaching Your Carry Bag

Install and hang the rings of bag onto the notches of seat tube extensions. Make sure all the rings have been securely locked on the notches before use.

The shopping/carry bag will fold with the walker when empty. There is no need for removal.



## Walker Tray

Your new walker comes with a tray which will allow you to carry items from room to room at home or areas where the terrain is smooth.

The tray simply sits on the seat of the walker with the four nibs protruding down through the four grommets in the seat. The nibs prevent the tray from sliding off the seat.



Caution should be exercised if drinks are being transported on the tray to avoid spillage. Drinks should be covered, unopened or in containers that have wide bases.

## Maintenance

Over a period of time the brake cables may stretch and need adjusting. If no further adjustment is possible the cables should be replaced.

Seek the help of your dealer when replacing brake cables.

The casters should be checked that they are secure each time before using the rollator.

Ensure the brakes operate correctly and the handles and hand knobs are secure.

Should any part of the rollator appear loose or insecure discontinue use until it can be inspected by an authorized dealer.

Check back strap for tears, fraying or other damage.

Check seat for damage or wear.

Check tires for damage or excessive wear.

## Periodic Maintenance Schedule:

### Weekly

- Check brake for malfunction
- Check all wheels and casters
- Check for loose bolts and nuts and knobs

### Monthly

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tires for wear or cracks
- Clean as required



## Yearly

Once a year take your rollator to an authorized repair centre for a full check, service and lubrication of all moving parts.

## Serviceable Components

Wheels	Hand Grips
Brake Cables	Brake Levers
Bearings	Knobs
Carry Bag	Bolts and Nuts

## Cleaning

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water or domestic anti-bacterial surface cleaners if required.

Care should be taken not to leave any cleaning residues on the handgrips or seat which may leave them slippery.

Do not use scouring pads or scrapers as this may damage surface coatings.

## Other Important Information

Maximum working limit 130kg.

DO NOT stand on this product.

Ensure both push handles are adjusted to the same height.

Always maintain a safe distance between your rollator and other pedestrians or objects.

Some components may cause harm or damage when they come into contact with other objects.

Do not attempt to replace the wheels or casters of your rollator. If they are faulty or require replacement please contact an authorized repairer.

Ensure that your rollator is fully unfolded before using.

Ensure that your fingers are clear from pivoting parts when folding.

Store in dry ventilated areas.

## Imported / Distributed by:

Factory Direct Medical  
34 Futurity Gate, Unit # 15  
Concord Ontario L4K 1S6  
Local: 416-739-8393  
Toll Free: 855-235-2400