

# EZee Life™

EZee Fold Ultra-Light Electric  
Folding Wheelchair  
CH4070

Controller Operating Guide



EZee Life™ Products

34 Futurity Gate, Unit 15, Concord, Ontario L4K 1S6

## Joystick



Carefully clean the joystick with a damp cloth & mild detergent. No corrosive or alcohol type cleansers.

### Power Switch Function:

- 1) Press the power button to turn the chair on. Indicator lights will light and horn will sound.
- 2) Press the power button again to turn off the chair. Indicator lights will go off and chair functions will cease.

### Horn Button:


Pressing the horn button will cause the horn to sound until released.

### Maximum Speed Button:

The maximum speed button will set the maximum speed of the wheelchair. The speed will be controlled by how far the joystick gimbal is moved (the further it is moved the faster the chair will move). Even at the fastest speed the chair will move slowly if the gimbal is moved very gently.

### Speed Indicator:

The speed indicator bar shows the speed the wheelchair is set at when the speed is adjusted. The higher the number with the lit indicator, the higher the maximum speed of the chair. The image above shows the chair is at speed 3.

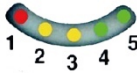

## Battery Indicator:

The same lights as the speed indicator are used for the battery level indicator. When the lights are coloured they show the charge level of the battery.

When all 5 lights are lit the chair is fully charged. The lights go out as the battery charge diminishes. When the red light is the only light lit, it is important to charge the chair as soon as possible.

If a long trip is planned it is important to charge before leaving even if the chair battery isn't fully discharged. It will not hurt the batteries to recharge occasionally when only partially discharged.

## Charging Off-Chair

When charging off-chair, plug the power cord into the wall socket and the charging cord into the battery box.

## Charging Information:

- Charger Input is AC 100 to 240 V, 50/60 Hz
- Battery is charged when light turns green
- Do not use wheelchair while charging (chair will not drive with charger plugged into it)
- During long term storage charge battery monthly even if not used.

## Seat Belt Use

Always use your seat while driving



Insert male end of seatbelt into female end until it clicks into place.

To undo, press sides of buckle and slide female end out.

## Other Information:

- Ensure wheelchair is not wobbly or has loose components before use.
- Ensure the battery is sufficiently charged prior to use.
- Drive a slow speed until comfortable with the chair's operation.
- Ensure chair is turned off while entering and exiting the wheelchair.
- Drive smoothly and don't exert too much pressure on joystick gimbal.

## Power/Manual Modes:

Electric wheelchairs will not roll when in the driving mode unless the joystick gimbal is moved. To push the chair manually it must be switched to the freewheeling mode.

To switch to freewheeling, pull the center of each rear wheel out and turn 90 degrees. To switch back, turn back and allow the center to move back inward.

The bottom picture shows the wheel in the freewheeling position.






## Unfolding:



To unfold your wheelchair grasp points A and B of the frame as shown and pull out point B.

Ensure the frame is fully opened and locked. Test to ensure the frame is latched and won't fold without unlatching.

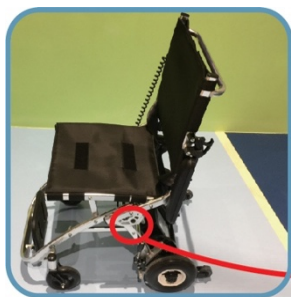

Grasp arms at their ends and rotate up and forward until they contact the stopper on the back post.




Flip down the foot pedal to the horizontal position



## Driving:

Ensure the power is turned off, sit in the seat and press the power button to activate the wheelchair. Press the joystick gimbal in the direction you wish to travel.

The further you push the joystick gimbal, the faster the wheelchair will respond. The brakes will automatically engage when the gimbal is released.

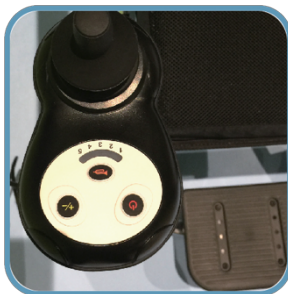

The joystick can be mounted on either the right or left arm.

HPU Medical Wholesale  
34 Futurity Gate, Unit # 15,  
Concord Ontario L4K 1S6  
Phone 416-739-1267  
Toll Free: 888-634-5808  
Fax: 416-739-7171