

# EZee Life™

EZee Fold Ultra-Light Electric  
Folding Wheelchair  
CH4070

## Quick Use Guide



EZee Life™ Products

34 Futurity Gate, Unit 15, Concord, Ontario L4K 1S6

## Battery Charging

The battery on the CH4070 EZee Fold can be charged either on the chair or off the chair. Charging off the chair allows the chair to be left in a vehicle, or in storage, while the battery is being charged.


### Charging On-Chair:

When charging on-chair, plug the power cord into the wall socket and the charging cord into the bottom of the joystick.



### Charging Off-Chair

When charging off-chair, plug the power cord into the wall socket and the charging cord into the battery box.

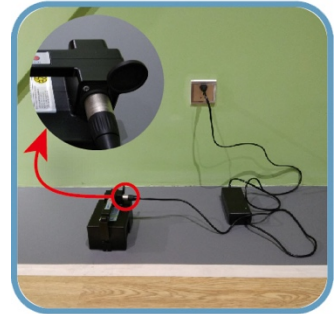

### Charging Information:

- Charger Input is AC 100 to 240 V, 50/60 Hz
- Battery is charged when light turns green
- Do not use wheelchair while charging (chair will not drive with charger plugger into it)
- During long term storage charge battery monthly even if not used.

## Seat Belt Use

Always use your seat while driving



Insert male end of seatbelt into female end until it clicks into place.

To undo, press sides of buckle and slide female end out.

## Other Information:

- Ensure wheelchair is not wobbly or has loose components before use.
- Ensure the battery is sufficiently charged prior to use.
- Drive a slow speed until comfortable with the chair's operation.
- Ensure chair is turned off while entering and exiting the wheelchair.
- Drive smoothly and don't exert too much pressure on joystick gimbal.

## Power/Manual Modes:

Electric wheelchairs will not roll when in the driving mode unless the joystick gimbal is moved. To push the chair manually it must be switched to the freewheeling mode.

To switch to freewheeling pull the center of each rear wheel out and turn 90 degrees. To switch back, turn back and allow the center to move back inward.

The bottom picture shows the wheel in the freewheeling position.







## Unfolding:




To unfold your wheelchair grasp points A and B of the frame as shown and pull out point B.

Ensure the frame is fully opened and locked. Test to ensure the frame is latched and won't fold without unlatching.

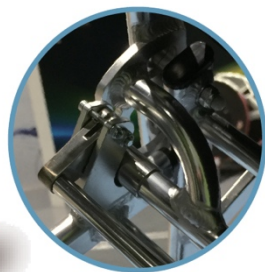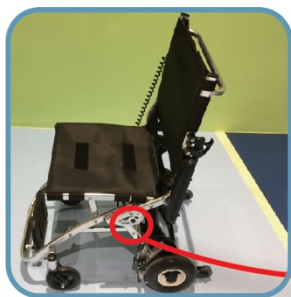

Grasp arms at their ends and rotate up and forward until they contact the stopper on the back post.

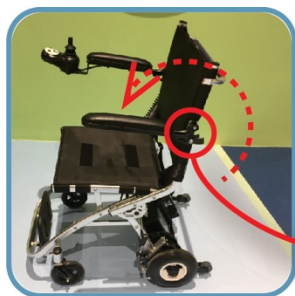


Flip down the foot pedal to the horizontal position



## Driving:

Ensure the power is turned off, sit in the seat and press the power button to activate the wheelchair. Press the joystick gimbal in the direction you wish to travel.

The further you push the joystick gimbal, the faster the wheelchair will respond. The brakes will automatically engage when the gimbal is released.

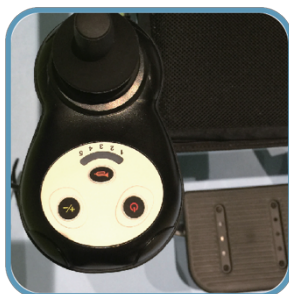

The joystick can be mounted on either the right or left arm.

HPU Medical Wholesale  
34 Futurity Gate, Unit # 15,  
Concord Ontario L4K 1S6  
Phone 416-739-1267  
Toll Free: 888-634-5808  
Fax: 416-739-7171