


## Remote Control Illustration

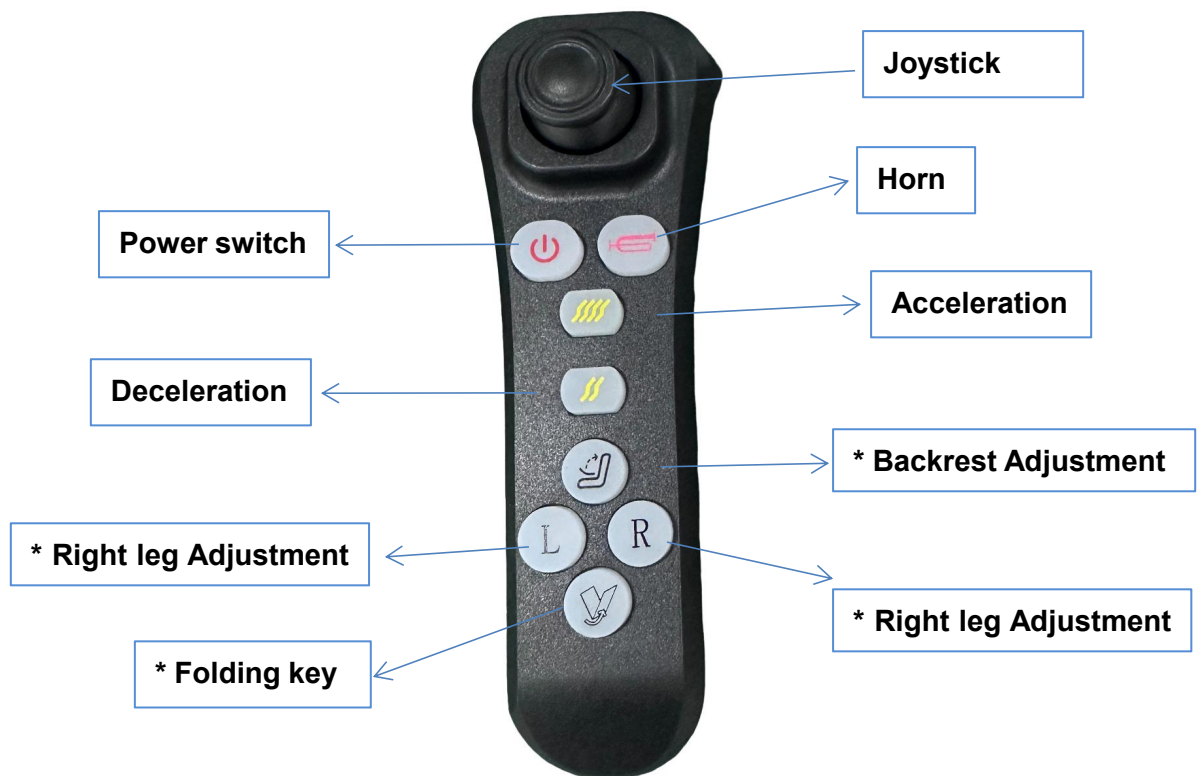

Note: with "\*" is for use in multifunctional electric wheelchairs.

## Instructions for use

### 1. First time use:

Open the battery compartment of the Remote Control and load 2pcs of NO.7 batteries correctly.

Turn on the remote control and controller switch respectively. The red light of the remote control switch key is on, press the “L” key and “R” key at the same time for 3 seconds and then it will be paired automatically.

The switch key turns blue indicating a successful connection.

### 2. No pairing required for second and subsequent use

3. If the bluetooth signal is disconnected or weak, the switch key indication will be red, and it will turn into blue if it is connected again, and you can move the wheelchair.

4. Turn off the remote control power switch, the remote control and controller will both be switched off.

5. Press the remote control horn button, the horn sounds

6. Press the remote control “Acceleration” key, increase the speed gear, the power wheelchair speed becomes faster

7. Press the remote control “Deceleration” key, lower the speed gear, the electric wheelchair speed becomes slower

8. Press the “Backrest Adjustment” key, the red light flashes, then press the “Acceleration” and “Deceleration” key to adjust the backrest angle and sitting position.

9. Press left leg adjustment key “L”, the red light flashes, then press “Acceleration” and “Deceleration” key can adjust left leg angle and posture.

10. Press right leg adjustment key “R”, the red light flashes, then press “Acceleration” and “Deceleration” key can adjust right leg angle and posture.

11. Press the wheelchair “folding key”, the red light flashes, then press the “Acceleration” and “Deceleration” key can adjust the unfolding or folding of the wheelchair.

## **Fault Resolution**

1. If the remote control does not open, please check whether the batteries are installed, whether the battery polarity is correct, and whether the battery power is sufficient.
2. The red light on the remote control stays on.
  - a. Check that the power wheelchair controller switch is on.
  - b. Whether the power wheelchair is within remote control range
  - c. Whether the remote control has been changed or not, if it has been changed, you need to pair it according to the “First time use”.
3. Note: Bluetooth control issue
  - a. If Bluetooth is not connected (the remote control lights up red) for 90 seconds, the remote control will automatically switch off;
  - b. If the Bluetooth is connected, but not operated, then they are all automatically switched off after 20 minutes.