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To fold, flip up the footplates or remove the footrests all together to make it lightweight Pull up on the seat upholstery folding the crossbraces (1)

To unfold, push down on the seat frame until the seat locks into place. (2)
Put the footrests on and/or fold down the footplates (after the person is seated)

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Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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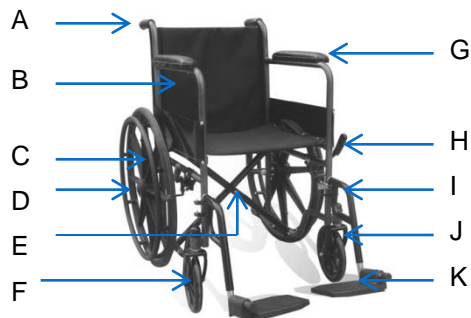
To swingaway the footrests, push the lever (1)
 To lock forward push back into transport position (2)
REMEMBER to help prevent falls, swingaway the footrests before transferring in or out of the chair

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Pull the brake handle back to lock and push forward to release (1 & 2)
REMEMBER to apply the brakes before getting in or out of the chair

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4 TAKING THE FOOTRESTS OFF AND PUTTING THE FOOTRESTS ON - RETIRER ET UTILISER LES REPOSE-PIEDS

Swing the footrest to the side by pushing the lever (1)
 Grab a hold of the top of the footrest and lift up (2) releasing the footrests from the pins. Repeat on the other side.

Holding the footrest in one hand, from the side align the pins with the holes in the footrest and attach (3)
 Swing footrest back in place.

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Using the wrench provided, place it over the bolt and turn to left to loosen or to the right to tighten in place (1) The footrest can slide up or down. Adjust the length so that the persons knees are parallel with the floor and the full thigh is supported by the seat.
REMEMBER Footplates need to keep at least a 2" clearance from the ground to overcome thresholds.
REMEMBER if the knees are too high the person may get a sore bum from increased pressure. If the knees are too low the person may slide out of the chair

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FOOTRESTS LOOK BACKWARDS? LES REPOSE-PIEDS SEMBLENT INVERSÉS?

Picture one (1) is what the footrests look like out of the box.
 Open the footplate (2)
 Twist the footplate 180 degrees, adjust the footrest length and tighten

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