

EZee Life[®]

Bath Lift User Manual

Model # CH2100



Table of contents

Product description	3
Components and parts	4
Pre-Operation	4
Don'ts	5
Charger	5
The charging process	5
Operation	6
Preparations before use.....	6
Before use.....	7
Set up and use.....	7
Emergency stop.....	8
Maintenance	8
Technical Data bath lift	9
Warranty	11
Troubleshooting	11

● Product description


Thank you for deciding to buy a product from Ezee Life. From now on, this lightweight, high-quality bath lift will make it a lot easier for you to get in and out of your bath.

The lift is built with high-quality, recyclable lightweight plastic. This makes our product easy to transport and stow away – and ideally suited for use in the home or away. The lithium-ion battery is in the hand control. The hand control itself can float in the water.

The drive motor and control electronics are completely watertight, so you can use it even when showering. The sturdy frame dismantles, enabling the seat to be raised or lowered from 8 cm to 48 cm (3.1 to 18.9 in) height. The lifting rod lifts and lowers the seat surface when the user pushes the up and down buttons. The drive unit with integrated backrest can be folded down for removal and automatically locks into the seat base when the lift is raised. This prevents unintentional tilting of the backrest. Intelligent power control ensures that the lift can only be lowered if sufficient power is available to raise it again.



● Components and parts

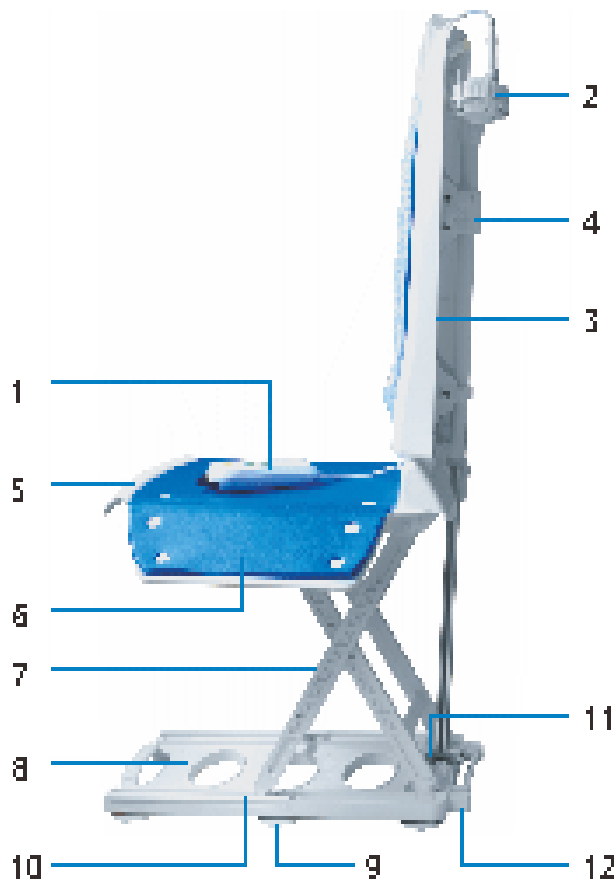

- Handset with battery and LED battery show low warning (1)
- Linear actuator (2)
- Back parts of chair (3)
- Safety pin (4)
- Seat parts of chair (5)
- Board parts (6)
- Support parts (7)
- Base parts (8)
- Suction feet (9)
- Slide track (10)
- Lift lock (11)
- Carry handle (12)
- Charger

Confirm all of above parts are included firstly while you get this bath lift, please contact your supplier to confirm and change promptly if there are any parts missing or damaged

● Pre-Operation

1. Before your first use, carefully read the instructions to familiarize yourself with the bath lift, it will make the operation simpler and more controllable.
2. Please keep these instructions in order that others can use them in the future.
3. Please charge the handset at least once a month when not being used regularly.
4. Assess the user to ensure sure that the lift is suitable for them; including user's weight, size, cognitive ability, joint range of motion. Please do not use the lift if there are any safety concerns and ensure there is a caregiver accompanying the user if there is any chance of them requiring assistance.
5. Designed for use in standard bathtubs and check with a professional to ensure the product is suitable.
6. Before each use, check to ensure the bath lift is not missing any parts or is damaged. Have a qualified technician or other experienced person check the lift over yearly.
7. Before each use, please check the suction cup feet and ensure the chair is

clean.

8. If using bath oils, bath salts or other bath supplies, please confirm they will not damage or gum up the sliding mechanism.
9. When in use, the user should sit down slowly, remain calm and avoid excessive or abrupt movement.
10. Only use the handset supplied with the lift.

● Don'ts

1. Don't exceed weight capacity (140 kg - 308.6 lb.)
2. Don't stand on the bath lift.
3. When using, always sit facing forward.
4. Don't use if the slope is too large (more than 5°).
5. Don't use if parts are damaged.
6. Don't use for other purposes than its intended use.
7. Don't let children use unaccompanied.
8. Don't allow body parts or other items into the slider/support area during use or maintenance.
9. Don't modify equipment.
10. Don't place objects in the scissors fork near (such as toothbrush, razor and so on).
11. Don't allow the handset to be near high heat sources.

● Charger

1. Please use dedicated charger.
2. The best battery capacity will be reached after several charging cycles.
3. When charging Please keep away water source, heat source and so on.
4. When the battery is running low, indicated by a red light, please charge the battery immediately, don't use continuously with low battery.
5. The charger light is red during charging, and green when fully charged; charging time is about 2.5 hours.
6. Charger is designed for the handset and can't be used for other purposes.
7. Don't open the handset.



● The Charging Process

1. Disconnect the handset contact from bath lift.


2. Make sure the connection and charger are dry.
3. Plug the charger's male plug into handset's female.
4. The charger plug into suitable wall socket.

Reverse the directions above when charging is finished.

● Operation - Preparations before use


Please confirm there are no parts damaged or missing before using.

1. Ensure the six suction feet are tightly installed. Tighten if needed.
2. Ensure there are no obstacles (such as shower head, tap, pipe fitting, etc.) in the bathtub that may prevent the bath chair from lifting up and lowering normally.
3. Confirm the bathtub meets the relevant standard.
4. Put the bath lift into the bathtub. If difficult, please get assistance.
5. Please ensure the ground is dry and no obstacles to prevent some accidents while moving the bath lift.
6. Please ensure there is enough space at the back of the bath chair in the bathtub.
7. Ensure six suction feet are well connected with the surface of the bathtub.
8. Ensure the male plug of bath lift and female plug of handset are totally dry before connecting.
9. Keep the wire of the handset free of the bath lift moving parts, in order to prevent it from being damaged or cutting the connection.

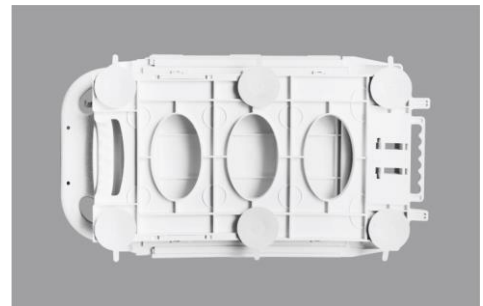

● Before Use

1. Before each use ensure that bath lift is operating normally and is safe for up/down use.
2. If the handset indicator lights, please charge it first.
3. Please ensure that the bath water temperature is not too hot or too cold.
4. Pressing the handset's up button will raise the seat, pressing the down button lower the seat, pressing the stop button will stop the lift.
5. Textured buttons will help users with poor eyesight.
6. After using the stop button press the up or down button to resume using the bath lift.
7. When lowered completely the bath lift can recline to 45°.
8. When raising the bath lift back adjust from recline to the upright position before the lift seat rises.

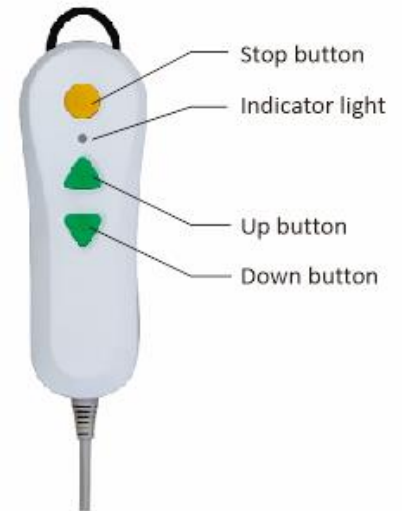

● Set Up and Use

- 1) Set the drive unit with integrated backrest into the bearing blocks of the seat
- 2) Hold the backrest by the hand grips and using the other hand tilt it backwards until the drive unit rests against the base plate.
- 3) Raise the bath lift up until the two side plates are level with and supported by the sides of the bathtub.
- 4) Sit on the outside of the side plate and back to the bathtub.
- 5) Keep on the above pose and then move to the center of the bath chair.
- 6) Swing your feet into the bathtub.
- 7) Press the down button on the handset to the needed position and ensure your hands and feet cannot be hurt in the process.
- 8) The bath lift will lower to the bottom of the tub before the backrest reclines to 45° by pressing the down button.
8. When finished, raise the bath lift up until the two side plates are supported on the edges of the bathtub, then move your feet out of the tub and stand.
9. Remove the bath lift from the tub if required.

● Emergency stop

1. If the bath lift doesn't stop when the buttons are released, please press the stop button for an emergency stop.
2. If you press the stop button when lowering, the down button will be locked and the lift must be raised up again to begin the lowering process again.

● Maintenance

1. Please rinse the bath lift with warm water after use, if needed you can use a cleaning cloth, brush and so on to clean and then wipe dry with a cloth.
2. Don't clean the handset (Part 1) and actuator (Part 2) using detergents or disinfectants, if needed use a damp cloth dipped in a small amount of cleaner on the handset and actuator. **Don't submerge in water.**



Make sure that the battery is fully charged before use! The voltage rating on the charging unit has to be compatible with the voltage of your power supply (e.g., 220V- 240V in Europe, and 110V in Canada / USA).

3. Attention should be paid to lift scissors to avoid pinching hands/fingers when cleaning or doing maintenance.
4. Please clean thoroughly between different users.
5. Ensure bath lift fully dry before storing.
6. If possible, please clean and maintain by hand.
7. Unplug the charger's power plug before cleaning.
8. Don't submerge or wet the charger, please clean with a dry cloth.



● Technical Data bath lift

Overall length (with backrest in a reclined position)	985mm (38.8in)
Overall width (seat incl. side flaps)	725mm (28.5in)
Width of seat	400mm (15.7in)
Depth of seat	500mm (16.7in)
Overall length of seat	590mm (23.2in)
Sitting height min	80mm (3.1in)
Sitting height max	487mm (19.2in)
Height of backrest	650mm (25.6in)
Width of backrest	350-360mm (13.8in-14.2in)
Backrest adjustment	5° - max 45°
Width of base plate	310mm (12.2in)
Length of base plate	612mm (24.1)
Load capacity max	140kg (308.6 lb)
Total weight (excl hand control)	9.6kg (21.2 lb)
Total weight (incl. hand control)	10.1kg (22.3 lb)
Weight of hand control incl. battery	0.5kg (1.1 lb)
Weight of drive unit	3.6kg (7.9 lb)
Weight of seat unit	6.0kg (13.3 lb)
Duration of lifting/lowering movement (105 kg load)	Lifting: 90 sec. / Lowering: 50 sec.
Operating voltage	14.8V
Storage battery(lithium-ion)	14.8V/2.2Ah
Automatic shutdown	max. 30min
Power input	16.8V



Initial position

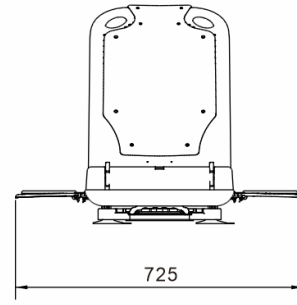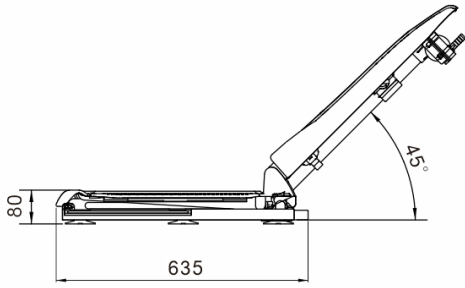

Highest height

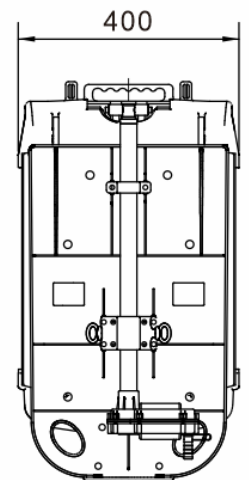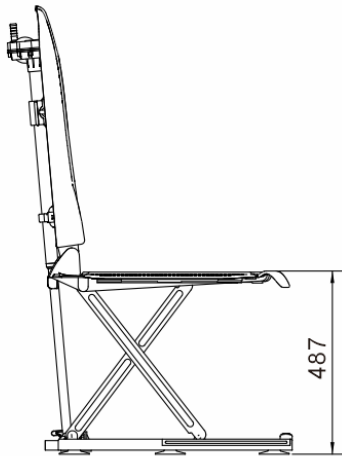

Folded size

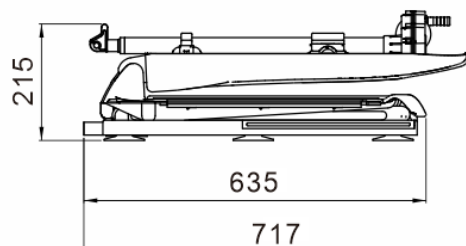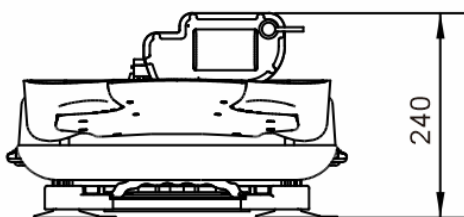

● Warranty

The warranty period of this product is 12 months (from the date of delivery - please keep purchase invoices/receipts).

In the event of a defect covered by this warranty, we will, at our option, repair or replace the item. Please contact your supplier/dealer for assistance should a defect become apparent.

The warranty does not extend to consequential costs resulting from the fault or defect in the product, in particular freight and travel costs, loss of earnings, expenses or other expenses which may occur.

The warranty does not cover normal wear and tear resulting from normal use or damage caused by accident, misuse, negligence or improper installation or assembly.


The vendor/manufacturer will not accept responsibility for any damages or injury caused by misuse, negligence or non-observance of the instructions supplied.

● Troubleshooting

Error	Possible cause	Remedy
Lift does not move	<ul style="list-style-type: none"> ● Battery is flat ● Contacts on hand control are contaminated ● Hand control is not connected properly 	<ul style="list-style-type: none"> -Charge battery -Unplug hand control, wipe contacts with a dry cloth and plug hand control in again. -Check plug type connections
Lift stops repeatedly	<ul style="list-style-type: none"> ● Battery is flat ● Excessive lifting load ● Jack plug not fully plugged 	<ul style="list-style-type: none"> -Charge battery -Reduce lifting load (note max load ratings, see information label on the bath lift) -Check jack plug connection
Battery does not charge	<ul style="list-style-type: none"> ● Not connected to mains power ● Contaminated contacts 	<ul style="list-style-type: none"> -Connect battery charger to mains power -Wipe contacts with dry cloth
Hand control does not function correctly	<ul style="list-style-type: none"> ● Jack plug not plugged in fully 	<ul style="list-style-type: none"> - Ensure jack plug is fully plugged in

Important remark concerning troubleshooting:

1. When bath lift's back is opened and the lower end of the actuator is mispositioned in picture 1, please adjust actuator to picture 2 by manually.

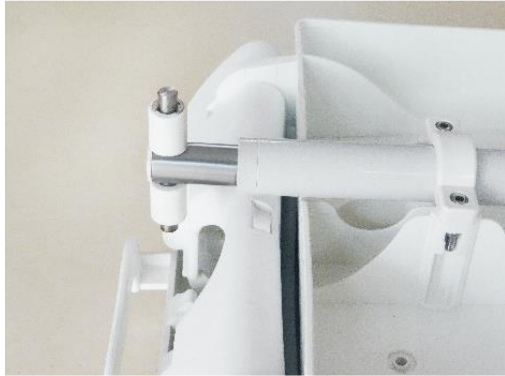

Picture1



Picture 2

4. If it is not possible to operate the bath lift despite observing all the above-mentioned information, please contact the specialist dealer who sold the device.

Imported and Distributed by:

**HPU Medical Wholesale
34 Futurity Gate, Unit 15
Vaughan, Ontario L4K 1S6**

**Phone: 416-739-1267
Toll Free: 888-634-5808
Fax: 416-739-7171**