

EZee Life™

Chrome Knurled Grab Bar

| Model # | Size |
|----------|------|
| CHRF2010 | 12" |
| CHRF2011 | 16" |
| CHRF2012 | 18" |
| CHRF2013 | 24" |
| CHRF2014 | 32" |



Floating flange allows for vertical, horizontal or angled installation.

Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

EZee Life™ Products

34 Futurity Gate, Unit 15
Concord (Ontario) L4K 1S6



Specifications:

- Wall/Bar Gap: 1½" between wall and gripping area
- Bar diameter: 1"
- Bar length: 12", 16", 18", 24" or 32"
- Installation method: Screwed to wall using rotating flange
- Weight capacity: Dependent on wall & installation quality

Features:

- Durable, chrome-plated steel
- Easy to clean
- No-slip knurled grip
- Mounts vertically, horizontally or angled
- 1½" between wall and gripping area
- Available in five lengths

Installation:

NOTE: Mounting flanges of grab bar must be aligned to mount to wall studs. Wood screws must go through the wall board or tile and attach to wood wall studs for solid support. For plastic tub and shower enclosures, drill through wall and mount grab bar to wall studs.

1. Place grab bar on wall in line with wall studs and mark location of screw holes. (Horizontal Attachment See Figure 1, Vertical Attachment See Figure 2.) After deciding on type of attachment, drill small hole to confirm the location of the stud.

WARNING: Both wood screws in each flange must be attached to wall studs or grab bar may pull out and serious injury could result

2. Use masonry bit to drill through tile or grout (Use caution when drilling near edges of tile. They may chip or shatter). For wood screws, drill small pilot hole into wood studs.

3. After holes are drilled, place grab bar on wall and attach with wood screw.

NOTE: Caulk around grab bar flanges to prevent moisture from seeping into walls.

4. Tighten all screws and bolts with screwdriver. Check periodically for tightness. Never use grab bar if any looseness is detected as it could result in personal injury.

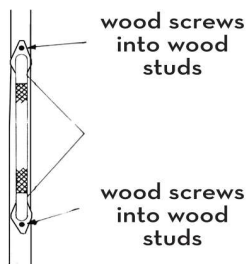


Figure 1

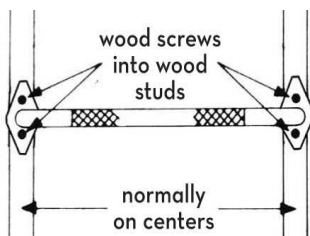


Figure 2